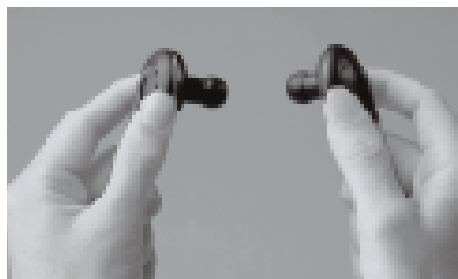
[the **stethoscope**]

**Users manual
Guarantee**

WelchAllyn
meditron

A**B****C****D****E****F****G****H**

WelchAllyn

Since 1915, Welch Allyn has been a company dedicated to bringing you the best in diagnostic instrumentation for the front line health care professional. The new Welch Allyn Electronic Stethoscope continues this tradition.

The Welch Allyn Electronic Stethoscope - based upon proprietary technology from Meditron AS - provides superior sound reproduction due to its sensor-based methodology. Other electronic stethoscopes depend upon a microphone for sound reproduction. Unfortunately, no matter the fidelity, microphones pick up ambient noise in the environment transmitted through the stethoscope diaphragm and chestpiece.

The Welch Allyn Electronic Stethoscope utilizes a highly directional, recessed pressure sensor which eliminates much of the ambient noise in the environment while faithfully reproducing body sounds in the same "color" as a traditional Welch Allyn stethoscope. Amplification can be varied as desired by the practitioner.

Recognizing the need for improved auscultation skills, the Welch Allyn Electronic Stethoscope is just one part of a complete teaching and diagnostic system. When combined with **[the distributor)** - a product for sending sounds from the stethoscope to others - and **[the analyzer)** - a software accessory for recording sounds and phono-cardiograms, the Welch Allyn Electronic Stethoscope System is a powerful addition to any hospital, clinic, or health care professional's diagnostic toolbox.

Table of contents

Overview

English	4 - 5
French	6 - 7
Spanish	8 - 9
German	10 - 11
Italian	12 - 13
Warranty	14 - 15
Product Diagram	16

[the**stethoscope**]

Congratulations on your choice of **[the**stethoscope**]** - an exciting, new sensor based technology for superior auscultation. This product provides new diagnostic capability as part of a complete auscultation system. It has been developed for all those who listen to body sounds with traditional stethoscopes.

The idea for the Welch Allyn / meditron electronic stethoscope is based on a new technological invention. Sounds are intercepted by a patented sensor in the receiver, which strengthens the signals and provides better sound reproduction than a traditional stethoscope.

The stethoscope is recommended for use directly against the skin, but also works when one is listening through a patient's clothing. It should not be used on wounds or broken skin. If it is, it must be cleaned immediately after use.

With optional accessories, the stethoscope can be linked to a PC so that sound signals can be visualized, stored, and later compared with former recordings of the patient. The sound picture, or phonocardiogram, can be transferred via modem and used in tele-medical environments.

The stethoscope is made of robust, durable materials and great emphasis has been placed on its design being user-friendly, hygienic, and ergonomic. It is equipped with an external port for a PC, a variable volume control, and a built-in filter for three predetermined frequency ranges:

1. Pulmonary (high frequency sounds)
2. Wide listening area
3. Cardiology

Cleaning

The stethoscope may be cleaned with a lightly moistened cotton wipe using chlorhexidine or alcohol. Avoid very high or low temperatures or contact with solvents. Do not immerse any of the stethoscope's components into liquid. Do not autoclave. Dry thoroughly prior to use if the unit is subjected to rain or high humidity.

Temperature

The operating range of the stethoscope is 5 to 40°C. Storage temperature is from -20°C to +60°C. After being stored in extreme temperatures for more than 24 hours, allow the stethoscope to return to normal operating temperature range prior to use.

Humidity

The stethoscope functions at +40°C, RH 90%, noncondensing.

Allergies

The stethoscope does not contain latex or emit any dangerous energy. To achieve the optimal benefit from the product, please study the following instructions carefully.

A On/off button

The stethoscope is turned on by pressing down the on/off button once. It is turned off by pressing down a second time; or, a timer will automatically turn off the stethoscope after 3 minutes.

B Choosing the frequency

The stethoscope has filters for three pre-determined frequency ranges. The filters are controlled with a small selector switch.

- 1. Pulmonary (H: High-frequency sounds)
- 2. Wide listening range (Mid-position)
- 3. Cardiology (L:Low frequency sounds)

C Volume control

The stethoscope is equipped with a variable volume control. Turning in the direction of the arrow reduces the volume.

D Headphones

The headphones are designed to adapt to the user's ears. Angle them slightly forward as you place them in your ears by gently twisting the speaker unit as shown in the picture. A good seal between the eartips and the ear canal are important for sound quality.

E Binaural

The headphones should grip your head comfortably, but must also provide a good seal so that the low-frequency sounds are not muffled. The spring tension can be adjusted to your personal preference by pushing the binaural tubes apart from each other several times.

F Ear tips

An extra set of ear tips is enclosed with the stethoscope. These are changed by holding

your thumb and index finger around the earpiece closest to the sound unit. Twist the earpiece carefully to the right and left, while also pulling it out of its groove. The new earpieces are put in place by twisting to the right and left while pressing in towards the sound unit.

Avoid covering the opening of the ear tips as this might harm the speakers.

G Batteries

The stethoscope is run on two high-energy, Lithium batteries: Model TL-5902 International reference size: 1/2 AA, ER 14252. In normal usage, the batteries have a lifetime of more than four years. New batteries can be ordered from your local dealer or bought at shops specialising in electronic equipment. The batteries are accessed by pushing the cover backwards as shown in the picture. Check that the batteries are put in with the correct polarity. The batteries are not re-chargeable.

H External attachments and accessories

An external cable port enabling additional products to be attached is shown in picture. The following accessory products are available from your Welch Allyn dealer:

Product Number	Description
5079-400	[the stethoscope]
5079-400 V	(Veterinarian)
5079-401	[the distributor]
5079-402	[the analyzer]
Accessories:	
5079-410	Batteries
5079-411	Battery Cover
5079-412	Eartips
5079-413	Distributor Cables
5079-414	Headphones

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English

Guarantee

Welch Allyn will remedy, through repair or replacement, all defects in the product that are due to errors in manufacture, materials, or construction and which become apparent within two years of sale. All claims must be made to your supplier without unreasonable delay.

Liability

Neither Welch Allyn nor Meditron AS is responsible for diagnoses, analyses, or outcomes resulting from the use of this equipment.

Service

Should any technical problems arise, please contact your dealer representative. Do not try to disassemble the product to carry out repairs.

Service may also be obtained by contacting the authorized Welch Allyn Service Centers below:

In the United States:

Welch Allyn Repair Dept. 95 Old Shoals Rd. Arden, NC 28704 USA

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In Canada:

Welch Allyn 160 Matheson Blvd - East, Unit 2, Mississauga, Ontario, Canada L4Z1V4

In Asia:

Welch Allyn 21-09 Golden Mile 6001 Beach Road, Singapore 199589, Rep of Singapore

In Australia:

Welch Allyn Ground Floor, 18-20 Orion Road, Lane Cove, NSW 2066, Australia

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Welch Allyn Repair Dept. 95 Old Shoals Rd. Arden, NC 28704 EE.UU.

En América Latina:

MDI, Repair Dept. 11300 NW 41st. Street, Miami, Florida 33178 EE.UU.

En Europa:

Welch Allyn GmbH. Postfach 31,
Zollerstrasse 2-4 D-72417,
Jungingen, Alemania

En Canadá:

Welch Allyn 160 Matheson Blvd - East,
Unit 2, Mississauga, Ontario, Canadá L4Z1V4

En Asia:

Welch Allyn 21-09 Golden Mile 6001 Beach Road,
Singapore 199589, Singapur

En Australia:

Welch Allyn Ground Floor, 18-20 Orion Road,
Lane Cove, NSW 2066, Australia

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In den USA:

Welch Allyn Repair Dept. 95 Old Shoals Rd. Arden,
NC 28704 USA

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In Kanada:

Welch Allyn 160 Matheson Blvd - East,
Unit 2, Mississauga, Ontario, Canada L4Z1V4

In Asien:

Welch Allyn 21-09 Golden Mile 6001 Beach Road,
Singapur 199589, Singapur

In Australien:

Welch Allyn Ground Floor, 18-20 Orion Road,
Lane Cove, NSW 2066, Australien

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Canada:

Welch Allyn 160 Matheson Blvd - East,
Unit 2, Mississauga, Ontario, Canada L4Z1V4

Asia:

Welch Allyn 21-09 Golden Mile 6001 Beach Road,
Singapore 199589, Rep of Singapore

Australia:

Welch Allyn Ground Floor, 18-20 Orion Road,
Lane Cove, NSW 2066, Australia

English

Declaration of Conformity

[the**stethoscope**), a part of the stethoscope system, conforms to the following standards or other normative documents:

EN 1441: 1997

IEC 601-1-2: 1993-04, with exception of all claims related to immunity (3V/m) ISO 10993-1: 1997-12-15.

CISPR 11: 1990-10 + amendment #1 & #2

Following the provisions of Medical Device Directive 93/42/EØF from 14th of June 1993.

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